



RoHS

### FEATURES

- Low Cost, Medical Applications
- Small Size and Reliable Performance
- Gel Isolation for Liquids
- Operates from 5°C to 40°C
- Compatible for Automated Assembly
- 1% Accuracy for Replacements
- 5.0 μV/V/mmHg Sensitivity
- Customization for OEM Applications

### **APPLICATIONS**

- Surgical Procedures
- Intensive Care Units
- Infusion Pumps
- Kidney Dialysis Machines
- Vacuum Assisted Birth
- Intrauterine Monitoring

# STANDARD RANGES



The 1620 is a fully piezo resistive silicon pressure sensor for use in invasive blood pressure monitoring. The sensor is designed to be used with automated assembly equipment and can be dropped directly into a customer's disposable blood pressure housing. The sensor is designed to meet the requirements as described in the Association for the Advancement of Medical Instrumentation (AAMI) specification for Blood Pressure Transducers.

The pressure sensor consists of a pressure sensing element mounted on a ceramic substrate. Thick film resistors on the ceramic substrate are laser trimmed for compensation and calibration. A plastic cap is attached to the ceramic substrate to provide an easy method of attachment to the customer's assembly and protection for the sensing element. A dielectric gel is placed over the sensor to provide electrical and fluid isolation.

The products are shipped on a tape and reel. Performance characteristics and packaging can be easily tailored on a special order basis to meet the requirements of specific customers.

#### SENSOR SOLUTIONS ///1620

1620

**SPECIFICATIONS** 

AAMI Specifications Low Cost Disposable Dielectric Gel Barrier

**Fully Tested & Calibrated** 

**Disposable Blood Pressure Sensor** 



## PERFORMANCE SPECIFICATIONS

#### Unless otherwise specified: Supply Voltage: 6.0 V<sub>DC</sub>, Ambient Temperature: 23°C

Unless otherwise specified: Supply Voltage: 6.0 PARAMETERS	MIN	TYP	МАХ	UNITS	NOTES
Operating Pressure Range	-50		300	mmHg	
Over Pressure	125			psi	
Zero Pressure Offset	-20		20	mmHg	
Sensitivity	4.95	5.00	5.05	μV/V/mmHg	
Calibration	97.5	100	102.5	mmHg	1
Linearity and Hysteresis (-30 to 100 mmHg)	-1		1	mmHg	2
Linearity and Hysteresis (100 to 200 mmHg)	-1		1	% Output	2
Linearity and Hysteresis (200 to 300 mmHg)	-1.5		1.5	% Output	2
Input Impedance					
1620-H	1200		3200	Ω	
1620-J	1200		3200	Ω	
1620-K	325		375	Ω	
Output Impedance	285		315	Ω	
Output Symmetry	0.95		1.05	Ratio	3
Supply Voltage	2	6	10	$V_{\text{DC}} \text{ or } V_{\text{AC}} \text{ rms}$	
Risk Current (@ 120 V <sub>AC</sub> rms, 60Hz)			2	μA	
Warm-Up Time		5		Seconds	
Frequency Response		1200		Hz	
Offset Drift			1	mmHg	4
Thermal Span Shift	-0.1		0.1	%/°C	5
Thermal Offset Shift	-0.3		0.3	mmHg/°C	5
Phase Shift (@ 5KHz)			5	Degrees	
Light Sensitivity (3000 Foot Candle)		1		mmHg	
Defibrillator withstand (400 joules)	5			Discharges	6
Sterilization (ETO)	3			Cycles	7
Operating Temperature	5		40	°C	
Storage Temperature	-25		+70	°C	
Humidity (External)	10-90% (non-condensing)				
Operating Product Life	168 Hours				
Shelf Life	5 Years				
Dielectric Breakdown	10000V <sub>DC</sub>				
Media Interface	Dielectric Gel				
Volume Displacement	4.5 x 10 <sup>-4</sup> mm <sup>3</sup>				
Weight	0.5 grams				


#### Notes

1. Output of sensor with no pressure applied and a 150kΩ resistor placed across +SUPPLY to +OUTPUT.

- 2. Best fit straight line.
- 3. Defined as common mode symmetry between any output and supply terminal.
- 4. Over an 8 hour time period after a 10 second warm-up.
- 5. Over operating temperature range 5°C–40°C with respect to 23°C.
- 6. One discharge per minute performed by customer.
- 7. Sterilization performed by customer, compatible with ETO, GAMMA, or E-Beam sterilization.
- 8. After curing, meniscus of gel shall be flush to .035" [.89mm] below surface
- 9. This model is not suitable for surface mount operation.




## DIMENSIONS

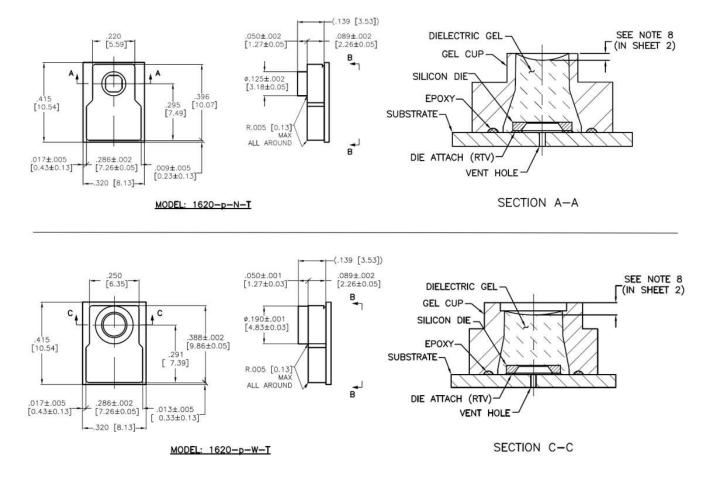

\*Dimensions indicated do not include flares.



### SHIPPING INFORMATION

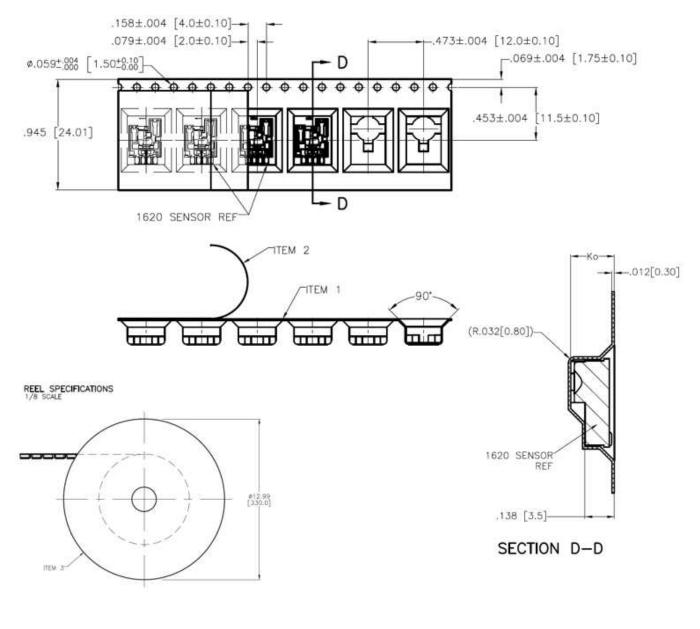

TAPE AND REEL INFO

A) MATERIAL:

ITEM 1, CARRIER TAPE: POLYCARBONATE


ITEM 2, COVER TAPE: POLYCARBONATE, HEAT PRESSURE SEAL

ITEM 3, PACKAGING TRAY: PLASTIC


- B). TOTAL PEEL STRENGTH SHOULD BE 10 TO 130 GRAMS.
- C). REFERENCE DOC: ANSI/EIA-481-C: 8mm THROUGH 200mm EMBOSSED CARRIER TAPING, 8mm AND 12mm PUNCHED CARRIER TAPING OF SURFACE MOUNT COMPONENTS FOR AUTOMATIC HANDLING.



### STANDARD PAD CONFIGURATIONS

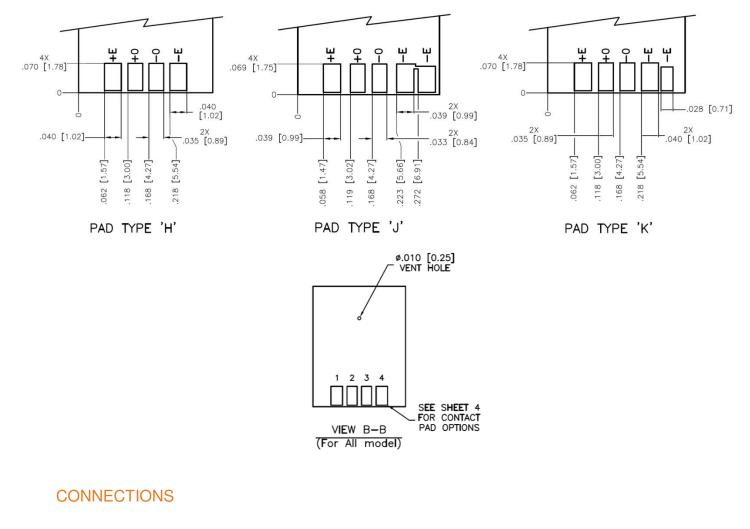


### EQUIVALENT CIRCUIT

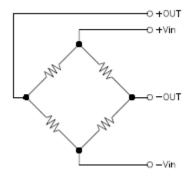



## **ORDERING INFORMATION**

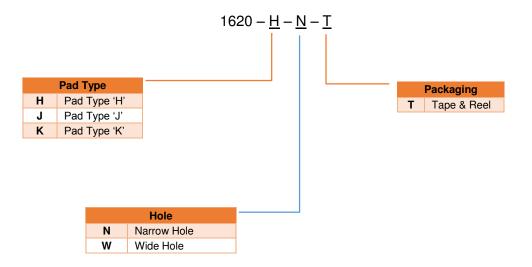



カタログに掲載してある製品の色は印刷インキの関係上、実際とは異なる場合があります。
製品のデザイン、仕様等などは、予告なく変更する場合があります。

本 社:〒124-0023 東京都葛飾区東新小岩3丁目9番6号 TEL:(03)3695-5431/FAX:(03)3695-5698 大阪支店:〒530-0054 大阪市北区南森町2-2-9(南森町八千代ビル7F) TEL:(06)6361-4831/FAX:(06)6361-9360 e-mail: sales-tokyo@krone.co.jp URL: https://www.krone.co.jp

Measurement Specialties, TE Connectivity, TE Connectivity (logo) and EVERY CONNECTION COUNTS are trademarks. All other logos, products and/or company names referred to herein might be trademarks of their respective owners.

The information given herein, including drawings, illustrations and schematics which are intended for illustration purposes only, is believed to be reliable. However, TE Connectivity makes no warranties as to its accuracy or completeness and disclaims any liability in connection with its use. TE Connectivity's obligations shall only be as set forth in TE Connectivity's Standard Terms and Conditions of Sale for this product and in no case will TE Connectivity be incidental, indirect or consequential damages arising out of the sale, resale, use or misuse of the product. Users of TE Connectivity products should make their own evaluation to determine the suitability of each such product for the specific application.

© 2015 TE Connectivity Ltd. family of companies All Rights Reserved.

株式会社 クローネ